AutoMini Time Delay

DANSK: Personlig slyngeforstærker til tv'et	side 2
ENGLISH: Personal loop amplifier for TV	page 9
FRANÇAIS: Amplificateur en boucle personnel pour télévision	page 16
NEDERLANDS: Persoonlijke ringleidingversterker voor TV	Pag. 23
NORSK: Personlig slyngeforsterker til TV-apparatet	side 30
SVENSKA: Personlig slingförstärkare till TV:n	sid 37

Ordering number: 200 21 82
AutoMini Time Delay Set, EU

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
Practical and simple loop amplifier

AutoMini is a personal amplifier for people with a hearing instrument. It amplifies and transmits sounds from, for example, a TV. The amplifier is easy to use and provides several connection options, both for picking up TV sound and listening. When it has been connected, you do not need to do anything when listening to the TV, the amplifier is always on. Set the hearing instrument to the T or MT position.


Because of its convenient shape and size and simple connection, it is easy to take the amplifier with you. This is practical when you want to be able to listen to TV, e.g. in a summer cottage, or at friends or in a hotel room. **Therefore make sure you keep this user manual and all the cords supplied!**

System components. The following parts are supplied:


Amplifier and mains adapter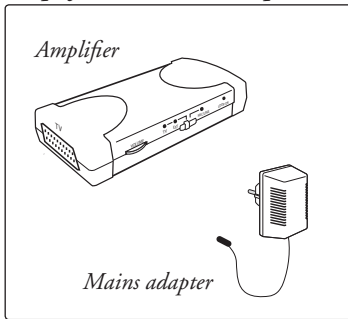

For picking up TV sound

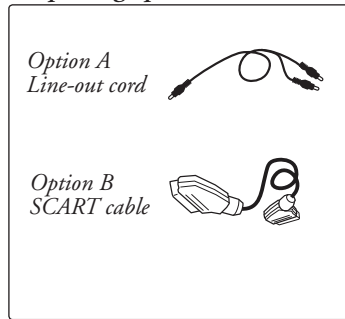
Listening loop

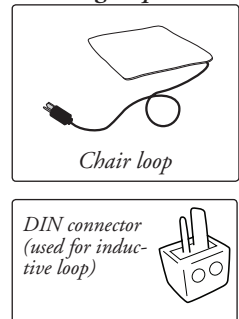Connection

1. Connecting the mains adapter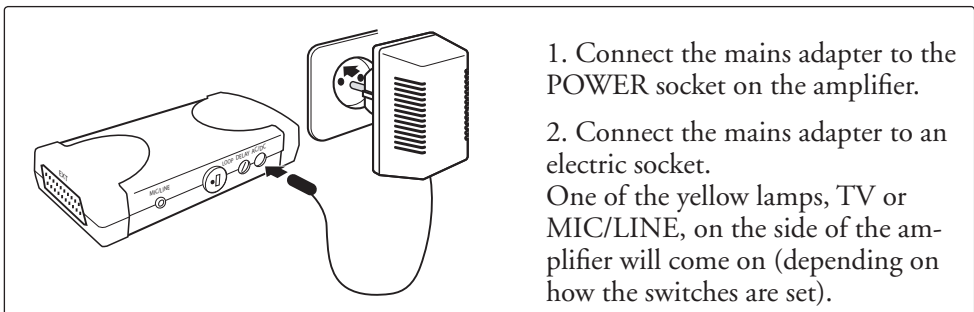

1. Connect the mains adapter to the POWER socket on the amplifier.

2. Connect the mains adapter to an electric socket.

One of the yellow lamps, TV or MIC/LINE, on the side of the amplifier will come on (depending on how the switches are set).

2. Picking up the TV sound

The amplifier should be connected to the TV. The amplifier works just as well if you already have one or more external units connected to the TV (e.g. video, DVD, digibox, satellite receiver). There are three different ways of transmitting the sound. For the best sound quality and simplest connection method, A is recommended as the preferred option, and then option B and then C.


Connection option A: Line Out


Check whether your TV has a Line Out output (usually on the rear of the TV, see TV user manual).

→ If your TV does *not* have a Line Out output, move on to connection option B.


Please note! Some TVs have a sound input on the front or on the side which looks like a Line Out output but which is NOT able to transmit sound to an amplifier. This is usually marked "Audio" and is located next to sockets marked "Video" and "S-Video" (see TV user manual).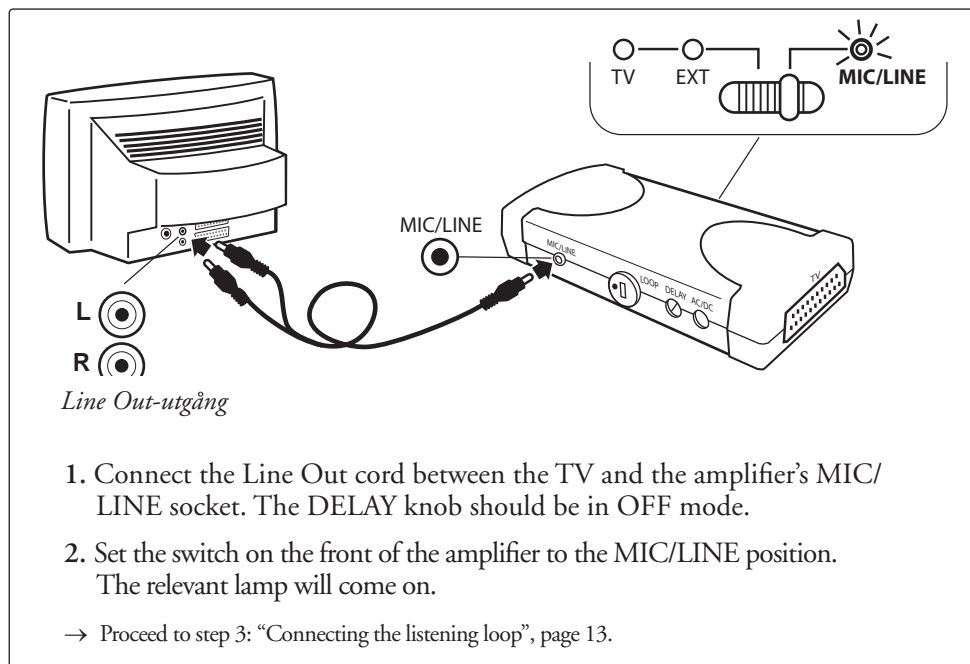

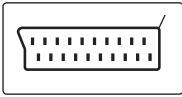Connection option B: SCART connector

Check whether your TV has a SCART socket. It does not matter whether it is in use.

→ If it does *not* have one, proceed to connection option C.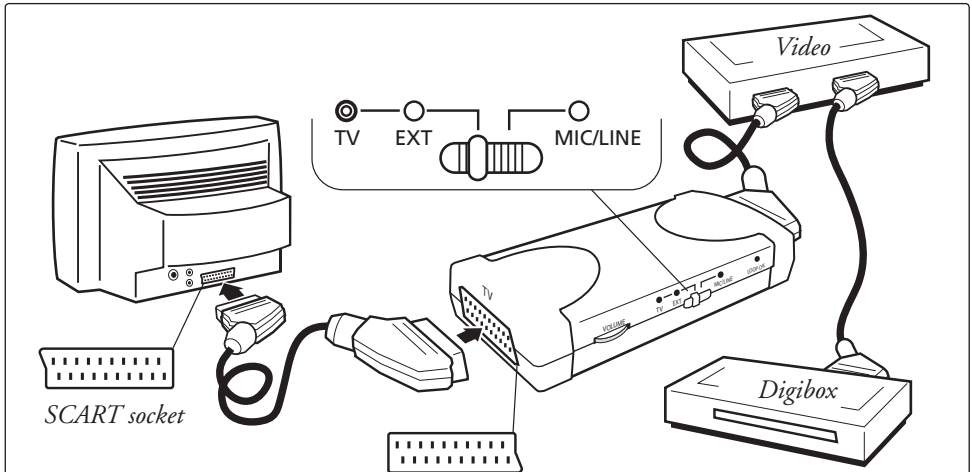

1. Connect the amplifier to the TV SCART socket using the SCART cord supplied. If the TV has more than one SCART socket, use the one that the TV manual recommends for video (usually SCART 1 or AV1).

If the TV's SCART socket is being used by one or more external units (e.g. video, DVD, digibox, satellite receiver), connect the amplifier between the TV and the external units. The units which are connected to the TV should be connected in series (see illustration above). The DELAY knob on the backside of AutoMini should be in OFF mode.


2. Set the switch on the front of the amplifier to the TV EXT position. One of the lamps, TV or EXT, will come on, depending on whether the TV or an external unit is being used at present.

→ Proceed to step 3: "Connecting the listening loop", page 13.

Please note! When correctly connected, the TV picture and sound will automatically be changed to the programme source you selected (e.g. video, DVD, digibox, satellite receiver). If you are watching video, DVD, satellite TV or another external unit and then switch to a standard analogue TV channel, in some cases you may, however, receive the wrong sound on the TV. In that case switch off the external unit, or all the external units, to obtain the correct sound. This phenomenon will never occur if you use connection options A or C.

Connection option C: Microphone


If the TV does not have a Line Out socket or SCART, use an FXT microphone (ordered separately) to pick up the TV sound.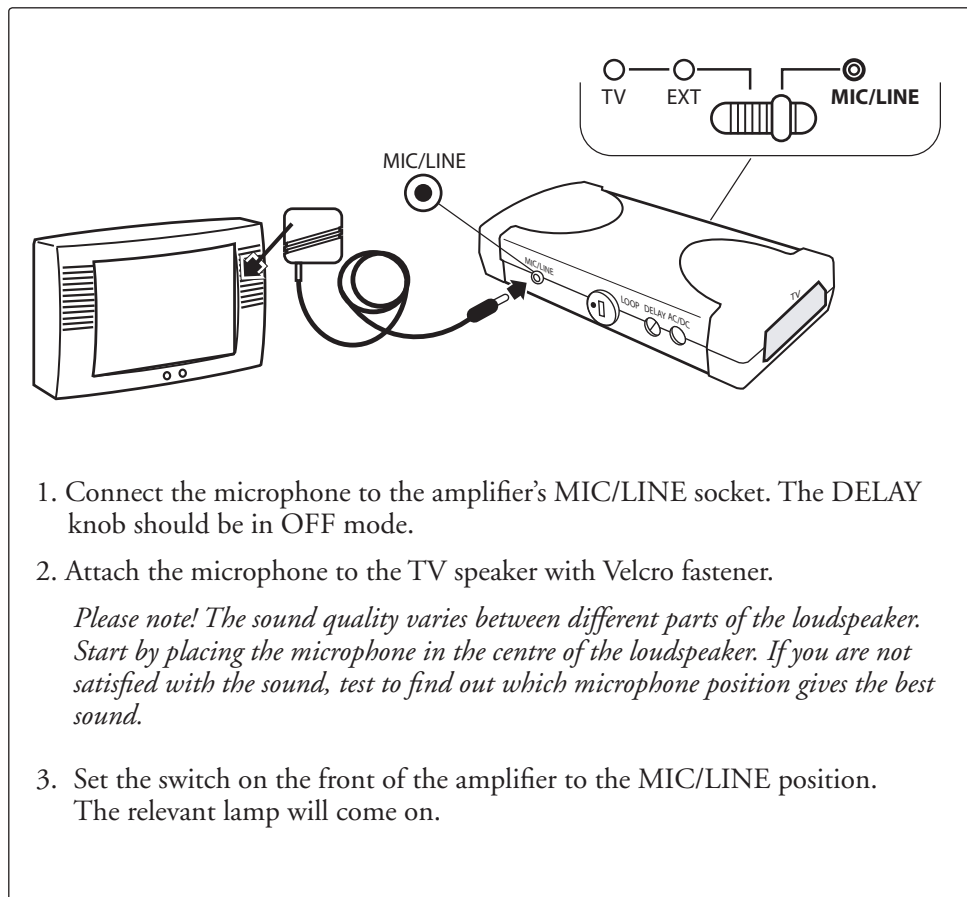

3. Connecting the listening loop

Select how you wish to listen to the sound. Wearers of hearing instruments can listen via the chair loop supplied or via our neck loop which can be purchased as an accessory.

Listening option A: Chair loop

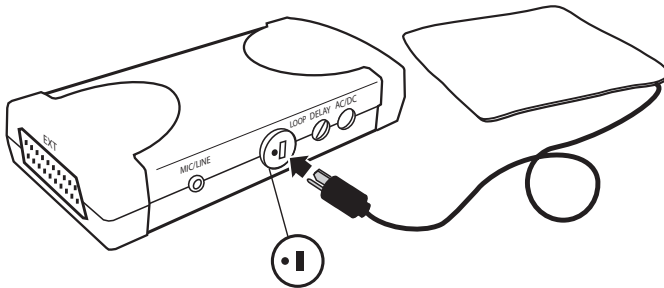1. Turn the volume down to minimum (0) so that the sound level is not too high at the outset. The DELAY knob should be in OFF mode.
2. Insert the chair loop connector in the amplifier's LOOP socket. The green LOOP OK lamp comes on.
3. Place the chair loop on the seat, either below or above the seat cushion.
4. Lay the cord supplied along the wall. Make sure that the cord is positioned so that you cannot trip up on it.
5. Set the hearing instrument to the T or MT position.
6. Adjust the volume control to a comfortable level and sit on the chair loop when listening.

Listening option B: Neck loop (accessory)

Our neck loop with volume control is an alternative to the chair loop. Sold separately. Fitted as set out in step 1 and 2 above and worn round the neck. Set the hearing instrument to the T or MT position.

Listening option C: Inductive loop of up to 20 m²

The loop wire must have a resistance of 0.5-1.5 ohms. Insulated copper wire with a cross section area of 0.28 mm² is recommended. Mount the supplied DIN connector that fits the LOOP socket. Follow points 1-6 above.

Synchronization of sound - the DELAY knob

Unsynchronized sound can be found in certain LCD or PLASMA TV sets. If the sound is not synchronized with the image (the sound precedes the image) you can adjust this with the knob marked "DELAY" on the backside of AutoMini. Increase the delay of the sound by turning the DELAY knob clockwise until you find that the sound is synchronized with the image. This should only be needed to do once.

4. Volume control

The amplifier has an automatic volume control (AGC = Automatic Gain Control), which means that the sound level in the loop remains constant, even if the sound level on the TV changes. If necessary, you can adjust the sound level using the volume control on the amplifier. You will normally only need to do this once.

Troubleshooting

Type of fault	Solution
There is no sound, or there is a humming sound:	<ol style="list-style-type: none"> 1. Check that the amplifier's mains cable is connected. 2. Check that the switch on the front of the amplifier is set correctly (see pages 10-12). 3. Test by moving the chair loop to another seat. (The hum may be due to electrical fields in the room.) 4. If you are using connection option A, Line Out, check that the socket is NOT a sound input for a video camera (see page 10 and/or the TV user manual).
The sound is faint:	<ol style="list-style-type: none"> 1. Increase the volume on the amplifier. 2. If you are using the microphone, check that it is attached to the TV speaker and that the TV volume is on. 3. Check that the hearing instrument is in the T or MT position. 4. Check that the chair loop is connected. Sit on the chair loop.
The LOOP OK lamp is off:	<ol style="list-style-type: none"> 1. Check that the chair loop is connected. 2. If the LOOP OK is off even though the chair loop is connected correctly, the chair loop is defective. See "Repairs" below.
The sound does not correspond to the TV picture:	<ol style="list-style-type: none"> 1. If the amplifier is connected in series with several external units, (e.g. video, DVD, satellite box) via the SCART input, switch off the other external units. 2. If you have cable TV with a digibox but are watching an analogue channel, switch off the digibox. 3. If you are watching a DVD film and have the right picture but the wrong sound, connect the Line-Out cord from the amplifier's MIC/LINE output to the DVD player's Line-Out output. 4. If you are recording on the video and watching another TV channel, use connection option C.
No sound is heard when a video camera is connected to the TV for playback.	If you are using connection option B (SCART cable), try out connection option A or C instead.
The sound precedes the image.	Adjust the DELAY knob until the sound is synchronized.

Cleaning and maintenance

Vacuum outside of amplifier and chair loop cushion using a soft brush. Use a damp cloth to wipe the outside of the amplifier. Never use cleaning products or solvents. From time to time check that the cords and connections are intact.

Repairs

If your product malfunctions, it must be repaired by a qualified technician. Do not attempt to open the product, since this would invalidate the warranty. (You may, however, remove the grey fabric cover from the chair loop). If your product requires service, please contact your hearing care professional for assistance.

Waste disposal

This product contains electrical or electronic equipment and should be disposed of carefully in the interests of your safety and the environment. Please contact your local hearing care professional concerning disposal of the product.

General security precautions

- During the installation: always start with the volume control set to minimum and then gradually increase the volume to a comfortable level. Hearing may be damaged if the sound level is too high.
- Chair loop: If you know that you have extremely sensitive skin, it may be good to avoid prolonged skin contact with the chair loop. Place the chair loop under the seat cushion or wear, for instance, trousers to avoid direct skin contact with the chair loop while sitting on it.
- Low-energy radiation (e.g. from burglar alarms, room surveillance systems, radio equipment and mobile phones) is not harmful to the product. These types of radiation could, however, momentarily affect the sound quality or create noise in the device
- Warning: External devices connected to the product must be safe according to the requirements of EN 60601-1-1, EN 60335-1 or EN 60065, as appropriate.

Part numbers and accessories

AutoMini Time Delay Set, EU 200 21 82

Included in AutoMini TD Set, EU:

AutoMini	200 10 32
Mains adapter	200 10 37
RCA cable with 3.5 mm jack, 1.5 m	200 10 33
SCART cable	200 10 39
Chair loop	200 20 45
DIN connector	16 337 300

Accessories:

Neck loop with volume control	414 01 60
Connection cord for neck loop	200 20 26
FXT microphone with 3.5 mm jack, 4 m	200 60 13

Technical data

AutoMini amplifier

Input signal:	50mV-12V
Output load:	4 Ohm
Power output:	6 W at 4 Ohm
Transient state time:	50 ms
Decay time:	35 s
Frequency range:	100 Hz-8 kHz, -3 dB rel. 1 kHz
Distortion:	<1%
Signal/noise ratio:	>40dB A-weighted
Delay of sound:	Off and 50-180 ms
Dimensions:	145 x 70 x 30 mm (with attachment approx. 38 mm)
Weight:	150 g (without mains adapter)

External mains adapter:

Plug-in adapter, primary 230V 50Hz, secondary 12V AC, 15VA

Chair loop:

Resistance: 4 ohms
 Dimensions: 41 x 41 x 0.2 cm
 Connector: 2-pin DIN loudspeaker plug
 Flex length: 10 metres

CE Any issues relating to the EU Medical Device Directive 93/42/EEC should be directed to GN ReSound A/S.

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6930 AB, Westervoort
INTERNATIONAL INQUIRIES:**GN ReSound AB ALD Division**