# Chair Loop

DANSK: Stoleslynge til slyngeforstærker .....	side 2
ENGLISH: Chair loop to loop amplifier .....	page 3
FRANÇAIS: Coussin magnétique vers amplificateur en boucle .....	page 4
NEDERLANDS: Luisterkussen naar ringleidingversterker .....	Pag. 5
NORSK: Stolslynge til slyngeforsterker .....	side 6
SVENSKA: Stolslinga till slingförstärkare .....	sid 7

Ordering number: 200 20 45

## Stoleslynge

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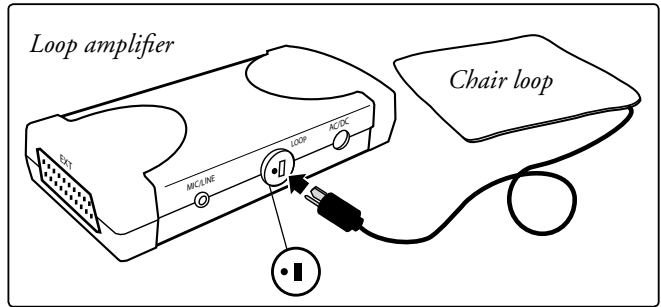
## Chair loop

This chair loop is designed for use together with our small and handy loop amplifier for transmitting TV sound. The chair loop is enclosed in a cushion. The attached cord is long enough to place along the wall.

### Connection

1. Connect the chair loop to the amplifier (see diagram or amplifier user manual).
2. Place the chair loop on the seat, either below or above the seat cushion.
3. Lay the cord supplied along the wall. Position the cord so that you will not trip over it.

### Listening

Set the hearing instrument to the T or MT position and sit on the chair loop when listening.

### Troubleshooting

Type of fault	Cause	Action
Sound is faint.	1. The hearing instrument is in the M position. 2. The volume on the loop amplifier is set too low. 3. The loop cushion is not connected.	1. Set the hearing instrument to the T or M/T position. 2. Adjust the volume to a comfortable level. 3. Connect the loop cushion.
Sound is faint even though volume on loop amplifier at maximum.	You are not sitting on the loop cushion.	You must sit on the loop cushion.

### Cleaning and maintenance

Vacuum the seat cushion using a soft brush. From time to time check that the cords and connections are intact.

### Repairs

If your product malfunctions, it must be repaired by a qualified technician. If your product requires service, please contact your hearing care professional for assistance.

### Waste disposal

This product contains electrical or electronic equipment and should be disposed of carefully in the interest of your safety and the environment. Please contact your local hearing care professional concerning disposal of the product.

### General precautions

- Low-energy radiation (eg from burglar alarms, room surveillance systems, radio equipment, mobile telephones) will not damage the device. They could, however, momentarily affect the sound quality or create noise in the device.
- Warning: External devices connected to the product must be safe according the requirements of EN 60601-1-1, EN 60335-1 or EN 60065, as appropriate.

### Technical data

Resistance: 4 Ohm  
 Dimensions: 41 x 41 x 0,2 cm  
 Pin: DIN  
 Length of cord: 10 m

 Any issues relating to the EU Medical Device Directive 93/42/EEC should be directed to GN ReSound A/S.

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**GN Resound A/S**
NL-6930 AB, Westervoort
**INTERNATIONAL INQUIRIES:****GN Resound AB ALD Division**