Bluetooth Solution Kit

Application guide for BT3030 and Neck Loop

Ordering number: 2002205

ENGLISH	page 3
DANSK	side 4
NEDERLANDS.....	Pag. 5
NORSK.....	side 6
SVENSKA	sidan 7

Pairing BT3030 and Neck Loop

The combination of BT3030 and Neck Loop facilitates mobile phone communication and listening to music/MP3 files (in case of MP3 player in mobile phone and support for Bluetooth Audio Profile) by sound transmission to your hearing instrument.

BT3030

Follow these steps to get started with the product.

1. Charge the BT3030 Bluetooth device according to instructions in the accompanying manual.
2. Attach the BT3030 Bluetooth device onto the lower part of the neck loop's blue ribbon with the aid of the enclosed clip ("clothing clip").
3. Connect the BT3030 Bluetooth device to the neck loop (the extension lead is not needed).
4. Set your hearing instrument in T mode.
5. For instructions on handling of the BT3030 Bluetooth device, see the accompanying manual "Quick Start Guide" or the "User Manual" on the CD.

Neck Loop

Accessories and order numbers:

Jabra A7010 Bluetooth hub (for fixed telephones)	2002208
Jabra A330 BT Dongle (for IP telephony)	2002207

Repairs

If your product malfunctions, it must be repaired by a qualified technician. Do not attempt to open the case of the device since this would invalidate the warranty. If your product requires service, please contact your hearing care professional for assistance.

Waste disposal

This product contains electrical or electronic equipment and should be disposed of carefully in the interest of your safety and the environment. Please contact your local hearing care professional concerning disposal of the product.

General precautions

- Low-energy radiation (e.g. from burglar alarms, room surveillance systems, radio equipment, mobile telephones) will not damage the device. They could, however, momentarily affect the sound quality or create noise in the device.
- Warning: External devices connected to the product must be safe according to the requirements of EN 60601-1-1, EN 60335-1 or EN 60065, as appropriate.

CE Any issues relating to the EU Medical Device Directive 93/42/EEC should be directed to GN ReSound A/S.

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GN ReSound AB
GN ReSound AB ALD Division
INTERNATIONAL INQUIRIES:**GN ReSound AB ALD Division**