Lynx™ Audio Detector


Ordering number: 200 30 77

ENGLISH: Lynx transmitter activated by acoustic signals.....	page 3
DEUTSCH: Durch akustische Signale ausgelöster Lynx-Sender.....	Seite 8
FRANÇAIS: Émetteur Lynx activé par signaux acoustiques.....	page 13
ITALIANO: Trasmettitore Lynx attivato da segnali acustic.....	Pag. 18
NEDERLANDS: Lynx-zender geactiveerd door akoestische signalen	Pag. 23
NORSK: Lynx-sender som startes av akustiske signaler	side 28
SVENSKA: Lynx-sändare som startas av akustiska signaler.....	sid 33


Audio Detector for the Lynx system

The audio detector is designed for use in conjunction with the Lynx – our wireless alert system for people with impaired hearing. The Lynx uses various transmitters and receivers that alert the user to noises such as a ringing telephone or doorbell, or the first signs of a fire.

The audio detector is a transmitter that can be used wherever it will be activated by specific sounds such as a ringing signal from an entry phone, a telephone or a doorbell. The signal is sent to the receiver via a wireless link. One or more repeaters may also be installed in order to amplify the sound signal and thus extend its range.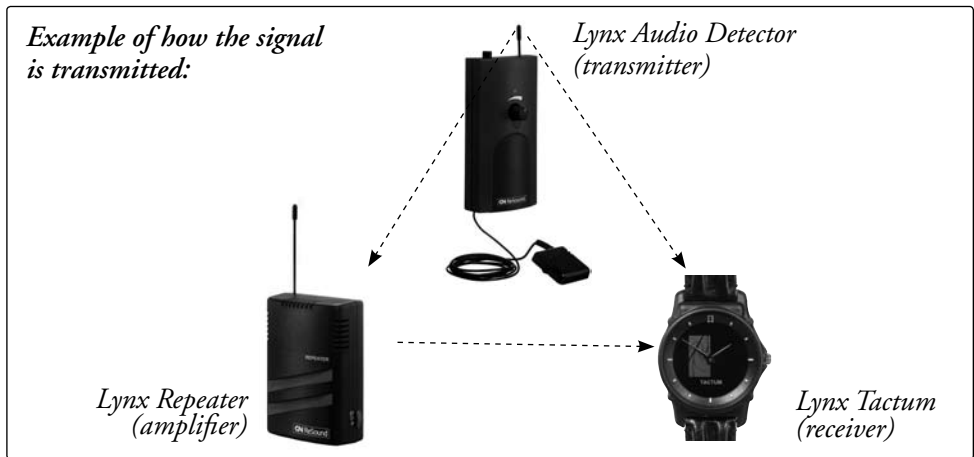


Overview diagram

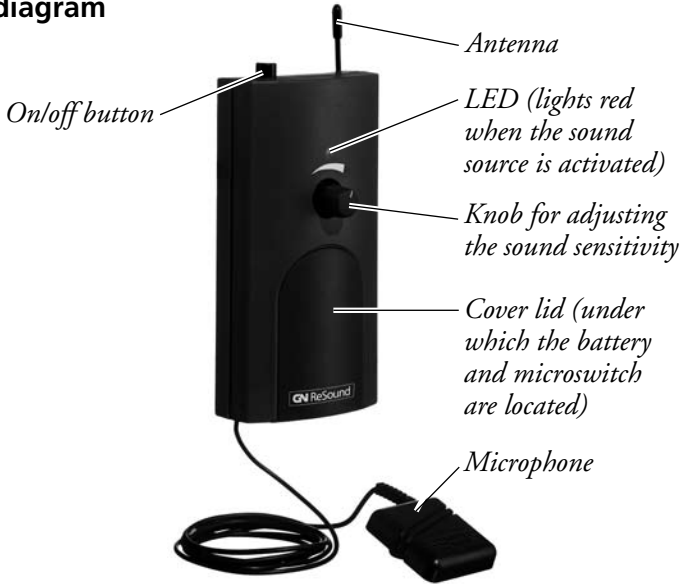Installing the audio detector

1. **Battery:** Slide the blue cover down and fit the battery provided.
2. **Select alarm symbol:** The audio detector is pre-installed to door. To change this default setting, see the next page.
3. **Microphone:** Place the microphone directly on the sound source or as close to it as possible to avoid false alarms. Check the correct location of the sound source. In a telephone set, for instance, the ringing tone can come from various openings under the set.

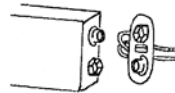The microphone flex has a length of 1.25 meters. If the distance to the sound source is shorter, the excess flex may be rolled up and placed under the cover. **NOTE:** In this case a groove must be cut in the bottom of the battery cover to avoid clamping the flex.

4. **Placing the audio detector:** The audio detector may be screwed permanently onto a wall, under a table or placed in some other suitable location.

Switching the audio detector on and off

Switching on: Press down the on/off button.

Switching off: Release the on/off button.

NOTE! To maximise the battery life, you should switch the detector off when it is not in use.

Adjusting the sound sensitivity

A knob on the front panel allows you to adjust the sound level to which the audio detector will respond. Start with the lowest sensitivity . Activate the sound source several times and turn the knob gradually clockwise until the transmitter starts. Check that the transmitter starts each time the sound source is activated. To avoid false alarms, the audio detector should be set to the lowest possible sensitivity.

When the sound source is activated

As soon as the sound source is activated, a wireless signal is sent to the receiver installed in the Lynx system. The red lamp on the audio detector simultaneously lights up for several seconds.

Depending on the type of receiver installed, the alarm takes the form of a vibration and a displayed symbol, a loud acoustic signal or a blinking light.

Read the user instructions for the installed receiver!

Battery level warning

The audio detector is powered by a battery. An alkali battery has a normal operating life of about three months and a lithium battery of about six months. When the battery begins to run down, the audio detector will display an appropriate warning while simultaneously sending a sound signal to the receiver in the alarm system.

Read the user instructions for the installed receiver!

Setting the sound symbol and changing the channel


The audio detector is equipped with a transmitter that sends signals wirelessly to the receiver installed in the Lynx system. The transmitter is located under the blue cover and contains an 8-pin microswitch with two functions:

- **Alarm symbol** (fire, telephone, door etc.) The audio detector is pre-set to door, but you may select any other suitable symbol if you wish. These symbols will depend on which receiver you use. If you wish to change the symbol, then set switches 6, 7 and 8 on the audio detector according to the table on the following page.
- **Channels** There are a total of 16 different channels (0-9, A-F). The system transmitter and receiver must be set to the same channel in order to communicate with each other. Channel 0 is reserved for a common fire alarm and should not be used. The audio detector and all the receivers in the Lynx system are pre-set to channel 9 and will interoperate perfectly when no changes are made.

Changing channel settings

It may sometimes be necessary to change the channel settings. If two or more users live close to each other, adjacent sounds may trigger the wrong system. To solve this problem, you can change the channel setting so that it is different for each user. Any of the free channels except for 0 may be selected. If you need to change a channel, set the switches on the audio detector as shown on the table below. In order to change a channel on the receiver, see the relevant user instructions.

Not used, must remain in off

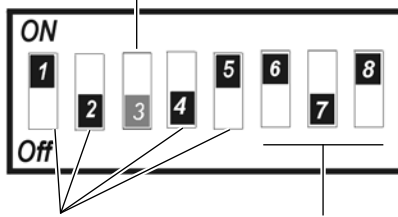Switches for channel

Switches for alarm symbol

Setting the microswitches for symbol selection:

	Switches:		
	6	7	8
Telephone	On	On	Off
Door	On	Off	On
Fire	Off	On	On
Baby	Off	Off	Off
Pager	On	Off	Off
Optional	Off	Off	On
Public noise	Off	On	Off

Setting the microswitches for channel selection:

Channel:	Switches:				
	1	2	3	4	5
0	Off	Off	Off	Off	Off
1	On	Off	Off	Off	Off
2	Off	On	Off	Off	Off
3	On	On	Off	Off	Off
4	Off	Off	Off	On	Off
5	On	Off	Off	On	Off
6	Off	On	Off	On	Off
7	On	On	Off	On	Off
8	Off	Off	Off	Off	On
9	On	Off	Off	Off	On
A	Off	On	Off	Off	On
B	On	On	Off	Off	On
C	Off	Off	Off	On	On
D	On	Off	Off	On	On
E	Off	On	Off	On	On
F	On	On	Off	On	On

Common channel for fire alarm

Pre-set channel on audio detector and all receivers

Troubleshooting

Type of fault	Action
The audio detector fails to detect the sound	1. Check that the battery is OK. 2. Check that the audio detector is switched on. 3. Check that the microphone is placed close to the sound source. 4. Check the set sound sensitivity (see page 4).
The audio detector gives a false alarm	1. Check that the microphone is placed close to the sound source. 2. Reduce the sound sensitivity (see page 4).
Receiver fails to respond	Check the range. Try to reduce the distance between the audio detector and the receiver or install a repeater to extend the range.

Cleaning and maintenance

Vacuum outside of audio detector with a soft brush several times per year. Use a lightly moistened cloth to dry off the exterior. Never use cleaning products or solvents.

Repairs

If the product malfunctions it must be sent to an authorised technician for repair. Do not attempt to open the case of the device since this would invalidate the warranty. If your product requires service, please contact your hearing care professional for assistance.

Waste disposal

This product contains electrical or electronic equipment and should be disposed of carefully in the interest of your safety and the environment. Please contact your local hearing care professional concerning disposal of the product.

General Precautions

- Low-energy radiation (eg from burglary alarms, room surveillance systems, radio equipment, mobile telephones) will not damage the device. They could, however, momentarily affect the sound quality or create noise in the device.
- Warning: External devices connected to the product must be safe according the requirements of EN 60601-1-1, EN 60335-1 or EN 60065, as appropriate.

Technical data

Frequency:	433.92 MHz
Modulation:	AM.
Battery:	One 9-V battery of type 6LR61 or IEC 6F22
Power consumption:	Stand-by: approx. 0.2 mA. Transmission: approx. 16 mA
Antenna:	Integrated (170 mm), partially concealed in the case
Range:	Approx. 100 m in a free field
Channels:	16.
Function code:	4 bits (3 bits selectable with microswitches)
Transmitter power:	<10 mW emission
Dimensions:	170 x 70 x 45 mm
Weight:	Approx. 200 grams.

Any issues relating to the EU Medical Device Directive 93/42/EEC should be directed to GN ReSound A/S.

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	Schakelaars:		
	6	7	8
Telefoon	On	On	Off
Deur	On	Off	On
Brand	Off	On	On
Baby	Off	Off	Off
Ander gezinslid	On	Off	Off
Facultatief	Off	Off	On
Extern geluid	Off	On	Off

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Kanaal:	Schakelaars:				
	1	2	3	4	5
0	Off	Off	Off	Off	Off
1	On	Off	Off	Off	Off
2	Off	On	Off	Off	Off
3	On	On	Off	Off	Off
4	Off	Off	Off	On	Off
5	On	Off	Off	On	Off
6	Off	On	Off	On	Off
7	On	On	Off	On	Off
8	Off	Off	Off	Off	On
9	On	Off	Off	Off	On
A	Off	On	Off	Off	On
B	On	On	Off	Off	On
C	Off	Off	Off	On	On
D	On	Off	Off	On	On
E	Off	On	Off	On	On
F	On	On	Off	On	On

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	Brytare:		
	6	7	8
Telefon	On	On	Off
Dörr	On	Off	On
Brand	Off	On	On
Baby	Off	Off	Off
Anhöriglarm	On	Off	Off
Valfri	Off	Off	On
Publikt larm	Off	On	Off

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Kanal:	Brytare:				
	1	2	3	4	5
0	Off	Off	Off	Off	Off
1	On	Off	Off	Off	Off
2	Off	On	Off	Off	Off
3	On	On	Off	Off	Off
4	Off	Off	Off	On	Off
5	On	Off	Off	On	Off
6	Off	On	Off	On	Off
7	On	On	Off	On	Off
8	Off	Off	Off	Off	On
9	On	Off	Off	Off	On
A	Off	On	Off	Off	On
B	On	On	Off	Off	On
C	Off	Off	Off	On	On
D	On	Off	Off	On	On
E	Off	On	Off	On	On
F	On	On	Off	On	On

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INTERNATIONAL INQUIRIES:**GN Resound AB ALD Division**
GN Resound A/S