Lynx™ Baby Alarm


Ordering number: 200 30 34

ENGLISH: Baby Alarm for our Lynx alert system	page 3
DEUTSCH: Babyalarm für das Lynx Alarmsystem.....	Seite 8
FRANÇAIS: Écoute-bébé conçu pour le système d’alerte Lynx	page 13
ITALIANO: Baby Control per il nostro sistema di allarme Lynx.....	Pag. 18
NEDERLANDS: Babyalarm voor het Lynx-systeem.....	Pag. 23
NORSK: Barnevakt til varslingssystemet Lynx	side 28
SVENSKA: Barnvakt till varseblivningssystemet Lynx.....	sid 33

Baby Alarm for the Lynx system

The Baby Alarm forms part of Lynx – our wireless alert system for people with impaired hearing. Lynx uses various transmitters and receivers that alert the user to noises such as a ringing telephone or doorbell, or the first signs of a fire.

The Baby Alarm reacts to the child crying and has a built-in transmitter that sends signals wirelessly to the receiver. One or more repeaters may also be installed in order to amplify the alarm signal and thus extend its range.

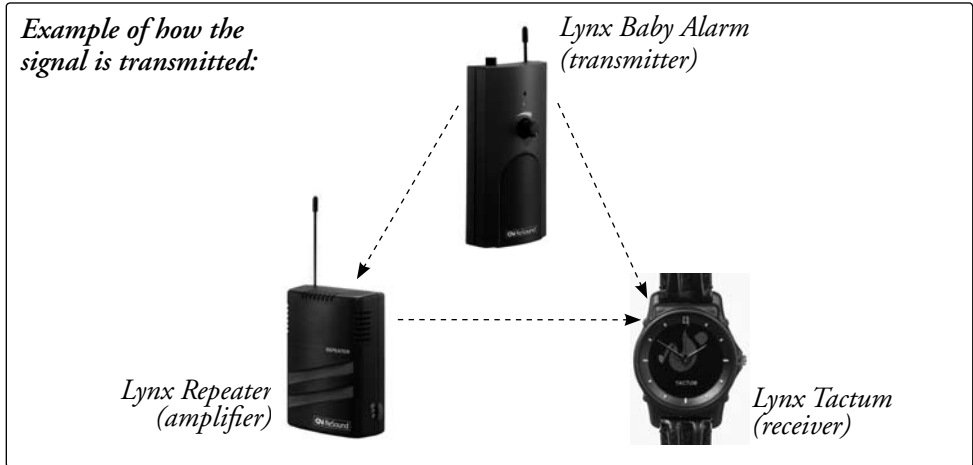
Overview

Installing the Baby Alarm

1. Open the blue cover and fit the battery supplied.

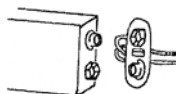2. Fit the Baby Alarm in the vicinity of the child's cot, out of reach of the child.

Switching the Baby Alarm on and off

Switching on: Press On/Off button briefly. The red LED flashes.

Switching off: Hold down On/Off button for four seconds until the red LED goes off. NOTE: to save the battery, switch off Baby Alarm when not in use.

Adjusting sound sensitivity

To make the alarm more sensitive to the baby's cries, increase sound sensitivity by turning the knob  on the front face clockwise. If the alarm reacts to noises other than the baby's cries, reduce sensitivity by turning the knob anti-clockwise.

When the child cries

When the child's cries exceed the set sound level, a wireless signal is sent to the receiver installed in the Lynx system. At the same time, the red lamp on the Baby Alarm lights up for a few seconds.

The alert is given by vibration and symbols, loud noise or flashing light, depending on the type of receiver.

Read the user manual for the receiver installed!

Battery warning

The Baby Alarm is powered by a battery. Normal service life for an alkaline battery is about 3 months, and for a lithium battery about 6 months. When battery power begins to fail, the Baby Alarm will send out an alert. It sends a flat-battery warning along with the alarm to the receiver in the alerting system.


Read the user manual for the receiver installed!

Pause

When the pause function is on, the Baby Alarm works as follows: When a “child crying” alarm has been sent out, there is then a four minute pause before another alarm is sent out. This provides an opportunity to attend to the child before another alarm is activated.


Switching on pause: Set microswitch 3 to Off (see below). The red LED flashes every four seconds to show that the Baby Alarm is set at pause.

When the Baby Alarm is set at pause, the pause period can be changed if desired. The pause period can be set to 4 (pre-set), 15 or 30 minutes. Press the On/Off button the required number of times as indicated below.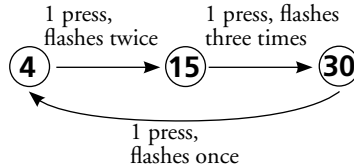

Switching off pause: Set microswitch 3 to ON (see below). Flashing stops after 4, 15 or 30 minutes, depending on pause setting when last switched off.

Changing the alarm symbol and the channel setting

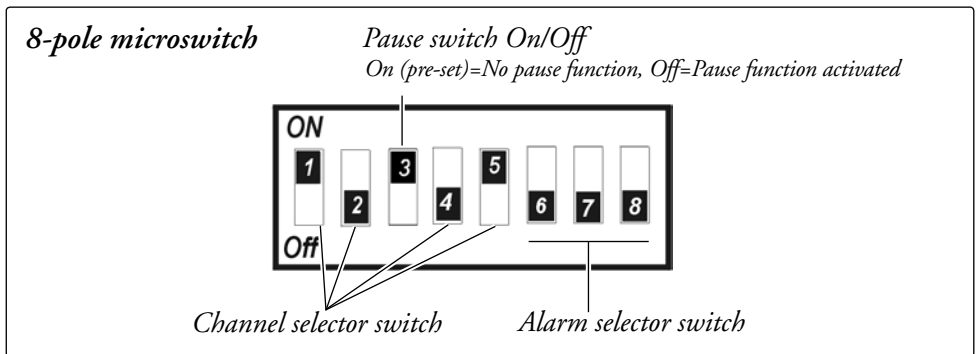The Baby Alarm is equipped with a transmitter that sends signals wirelessly to the receiver installed in the Lynx system. The transmitter is located under the blue cover and has an 8-pole microswitch that is used for two purposes:

- **Alarm symbol** (fire, telephone, door, etc). The alarm symbol on the Baby Alarm is pre-set to Baby Alarm. **NOTE: If Lynx MiniVib is used as a receiver, the alarm symbol on the Baby Alarm must be changed, see next page.**
- **Channels.** There is a total of 16 different channels (0-9, A-F). Transmitters and receivers in the system must be set to the same channel so that they can communicate. Channel 0 is reserved for a common fire alarm and should not be used. The Baby Alarm and all receivers in the Lynx system are all pre-set to channel 9, and then act together without alteration.

Changing the alarm symbol when using MiniVib

Set the switches on the Baby Alarm as shown in the table below. The Alarm Call/Pager alarm symbol is set as an Alarm Call on MiniVib.

Setting the microswitches for symbol selection:

Alarm Symbol:	Switch:		
	6	7	8
Telephone	On	On	Off
Door	On	Off	On
Fire	Off	On	On
Baby	Off	Off	Off
Alarm call/ Pager	On	Off	Off
Optional Public alert	Off	Off	On
	Off	On	Off

Pre-set alarm symbol

Alarm symbol for use with MiniVib

Changing the channel setting

In some cases, it may be necessary to change the channel setting. If two or more users live near one another, the alarms can interfere. This can be rectified by changing the channel settings so that they are different for everyone. Any channel can be selected except Channel 0.

If you wish to change channel, set the switches for the Baby Alarm in accordance with the table below. To change channels on the receiver, see user manual.

Setting the microswitches for channel selection:

Channel:	Switch:				
	1	2	3	4	5
0	Off	Off	On	Off	Off
1	On	Off	On	Off	Off
2	Off	On	On	Off	Off
3	On	On	On	Off	Off
4	Off	Off	On	On	Off
5	On	Off	On	On	Off
6	Off	On	On	On	Off
7	On	On	On	On	Off
8	Off	Off	On	Off	On
9	On	Off	On	Off	On
A	Off	On	On	Off	On
B	On	On	On	Off	On
C	Off	Off	On	On	On
D	On	Off	On	On	On
E	Off	On	On	On	On
F	On	On	On	On	On

Common channel for fire alarm

Pre-set channel on Baby Alarm and all receivers

Troubleshooting

Type of fault	Action
Baby Alarm does not detect sound.	1. Check that the battery is OK. 2. Check that the Baby Alarm is switched on. 3. Check the sensitivity setting. 4. Switch off pause, if applicable (see page 5).
Baby Alarm gives a false alarm.	1. Check sensitivity setting (reduce if necessary). 2. If the receiver is a MiniVib, check that the alarm symbol on the Baby Alarm is set to Alarm Call/Pager (see page 6).

Cleaning and maintenance

Vacuum outside of Baby Alarm with a soft brush several times per year. Use a lightly moistened cloth to dry off the exterior. Never use cleaning products or solvents.

Repairs

If the product malfunctions it must be sent to an authorised technician for repair. Do not attempt to open the case of the device since this would invalidate the warranty. If your product requires service, please contact your hearing care professional for assistance.

Waste disposal

This product contains electrical or electronic equipment and should be disposed of carefully in the interest of your safety and the environment. Please contact your local hearing care professional concerning disposal of the product.

General Precautions

- Low-energy radiation (eg from burglary alarms, room surveillance systems, radio equipment, mobile telephones) will not damage the device. They could, however, momentarily affect the sound quality or create noise in the device.
- Warning: External devices connected to the product must be safe according to the requirements of EN 60601-1-1, EN 60335-1 or EN 60065, as appropriate.

Technical data

Frequency:	433.92 MHz.
Modulation:	AM.
Battery:	1 x 9 V type 6LR61 or IEC 6F22.
Power consumption:	Standby: approx. 0.2 mA. Transmitting: approx. 16 mA.
Aerial:	Integrated (170 mm), partially concealed in cover.
Range:	Up to 30 m indoors.
Channels:	16.
Function code:	4 bit (3 bit selectable via microswitches).
Transmitter power:	<10 mW radiation.
Dimensions:	170 x 70 x 45 mm.
Weight:	approx. 200 g.

Any issues relating to the EU Medical Device Directive 93/42/EEC should be directed to GN ReSound A/S.

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<i>Alarm-symbool:</i>	<i>Dip-switches:</i>		
	6	7	8
Telefoon	On	On	Off
Deurbel	On	Off	On
Brand	Off	On	On
Baby	Off	Off	Off
Noodoproep/ Pager	On	Off	Off
Optioneel	Off	Off	On
Openbaar alarm	Off	On	Off

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<i>Kanaal:</i>	<i>Dip-switch:</i>				
	1	2	3	4	5
0	Off	Off	On	Off	Off
1	On	Off	On	Off	Off
2	Off	On	On	Off	Off
3	On	On	On	Off	Off
4	Off	Off	On	On	Off
5	On	Off	On	On	Off
6	Off	On	On	On	Off
7	On	On	On	On	Off
8	Off	Off	On	Off	On
9	On	Off	On	Off	On
A	Off	On	On	Off	On
B	On	On	On	Off	On
C	Off	Off	On	On	On
D	On	Off	On	On	On
E	Off	On	On	On	On
F	On	On	On	On	On

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	Brytare:		
	6	7	8
Larmsymbol:			
Telefon	On	On	Off
Dörr	On	Off	On
Brand	Off	On	On
Babylarm	Off	Off	Off
Väckning/ Anhörigalarm	On	Off	Off
Valfri	Off	Off	On
Publikt larm	Off	On	Off

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Kanal:	Brytare:				
	1	2	3	4	5
0	Off	Off	On	Off	Off
1	On	Off	On	Off	Off
2	Off	On	On	Off	Off
3	On	On	On	Off	Off
4	Off	Off	On	On	Off
5	On	Off	On	On	Off
6	Off	On	On	On	Off
7	On	On	On	On	Off
8	Off	Off	On	Off	On
9	On	Off	On	Off	On
A	Off	On	On	Off	On
B	On	On	On	Off	On
C	Off	Off	On	On	On
D	On	Off	On	On	On
E	Off	On	On	On	On
F	On	On	On	On	On

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INTERNATIONAL INQUIRIES:**GN ReSound AB ALD Division**