# Lynx™ Flash


Ordering number: 200 30 78

ENGLISH: Flash alert for the Lynx alert system.....	page 3
DEUTSCH: Blitzleuchte für das Lynx Alarmsystem .....	Seite 8
FRANÇAIS: Alerte clignotante du système Lynx.....	page 13
ITALIANO: Flash alert per il sistema di allarme Lynx.....	Pag. 18
NEDERLANDS: Flitslamp voor Lynx alarmsysteem .....	Pag. 23
NORSK: Flash-varsling for Lynxvarslingssystemet.....	side 28
SVENSKA: Blinklampett till varseblivningssystemet Lynx.....	sid 33

## Optical alerting with the Lynx Flash



The flash alert forms part of Lynx – our wireless alert system for people with impaired hearing. The Lynx uses various transmitters and receivers that alert you to noises such as a ringing telephone or doorbell, or the first signs of a fire.

The flash alert is a wireless receiver which uses a powerful flashing halogen lamp to alert the user when one of the transmitters in the Lynx system is activated. The various transmitters can be recognised by the different flash frequencies of the halogen lamp and by the different coloured LEDs. The transmitters send signals wirelessly to the flash alert. One or more repeaters may also be installed in order to amplify the alarm signal and thus extend its range.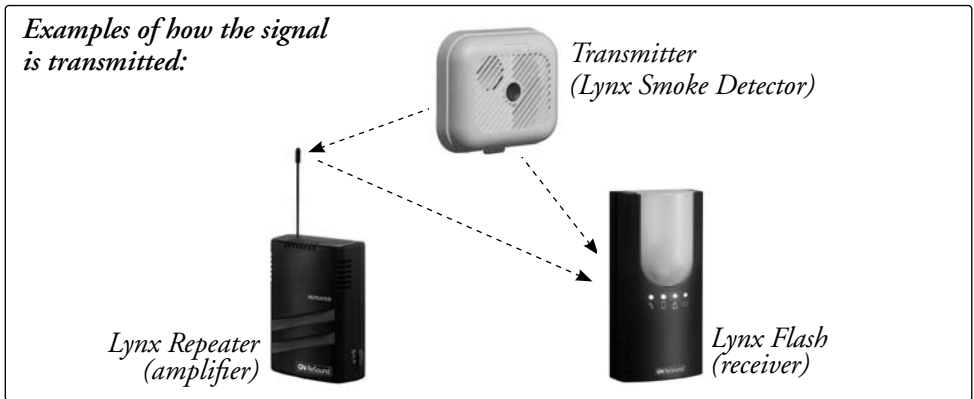

## Overview

*Under the lamp cover:*  
Channel selector  
(microswitches)

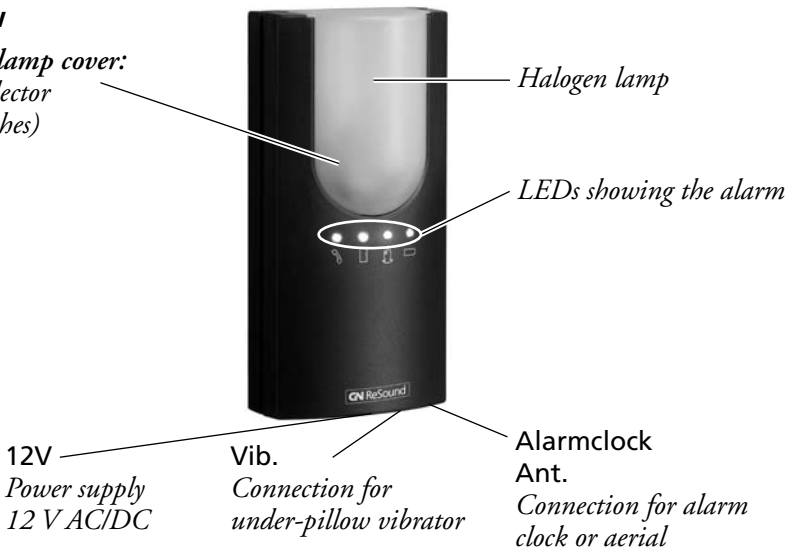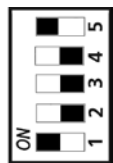## Location and fitting

It may be advisable for several flash alerts to be fitted in the house. The recommended location is somewhere clearly visible in the room such as on the TV, or on the bedside table if an under-pillow vibrator or alarm clock is used. The flash alert can be fitted to the table stand supplied or screwed onto the wall.

## Power supply

Connect the flash alert to the mains, with the mains adapter supplied. The four LEDs will light up one at a time to indicate that the flash alert is ready for use. *The flash alert will retain the previous alert until the next alert is received.*

**NOTE: The flash alert is supplied only by mains power and does not have a battery back-up.**

## External aerial

When the flash alert is used without an alarm clock, the external aerial supplied is connected to the Alarm clock Ant. connection. (see Overview on previous page). This extends the range of the flash alert. If an alarm clock is connected, the alarm clock cable acts as an aerial.

*External aerial*

## Function test

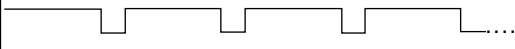
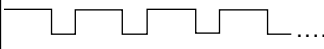
1. Start by installing the transmitters that are to be used.

**Read the user manual for the transmitters installed!**

2. Test the system by activating the transmitters in the system one at a time. Check that the correct alarm is indicated (see next page). The previously indicated alert is retained until the next alert is received.

### Alarm indication

The flash alert can receive an alarm from up to seven different transmitters. The halogen lamp flashes according to a certain pattern and the LEDs for the relevant alarm come on.

Alarm type:	LED:	Flash frequencies:
<b>Telephone</b>	orange - - -	
<b>Door</b>	- green - -	
<b>Fire</b>	- - red -	
<b>Baby</b>	orange green red -	
<b>Pager</b>	orange green - -	
<b>Optional</b>	- green red -	
<b>Public alert</b>	orange - red -	

### Battery warning

The flash alert also indicates when the battery power of the transmitters begins to fail. The transmitters send a flat-battery warning along with the alarm to the flash alert. Initially the alarm is shown as usual, then the yellow battery lamp flashes (until the next alarm is received).

## Changing channels

The transmitters in the Lynx system transmit the signal wirelessly to the flash alert. They can transmit on a total of 16 different channels (0-9, A-F). The flash alert and all transmitters must be set to the same channel so that they can communicate. They are pre-set to channel 9, and then operate together without alteration. In some cases, it may be necessary to change the channel setting. If two or more users live near one another, the alarms can interfere. This can be rectified by changing the channel settings so that they are different for everyone. Any channel can be selected except Channel 0. (Channel 0 is reserved for a common alert such as a fire alarm.)

If you need to change the channel on the flash alert, proceed as follows: Move the white lamp cover upwards. You can see where the microswitches are located from the diagram on page 3. Set the switches as shown in the table below. To change channels on the receiver, see user manual.

*Table showing microswitch settings:*

Channel:	Switch:					
	1	2	3	4	5	
<b>0</b>	Off	Off	Off	Off	Off	— Common channel for fire alarm
<b>1</b>	On	Off	Off	Off	Off	
<b>2</b>	Off	On	Off	Off	Off	
<b>3</b>	On	On	Off	Off	Off	
<b>4</b>	Off	Off	Off	On	Off	
<b>5</b>	On	Off	Off	On	Off	
<b>6</b>	Off	On	Off	On	Off	
<b>7</b>	On	On	Off	On	Off	
<b>8</b>	Off	Off	Off	Off	On	
<b>9</b>	On	Off	Off	Off	On	— Pre-set channel on flash alert and all transmitters
<b>A</b>	Off	On	Off	Off	On	
<b>B</b>	On	On	Off	Off	On	
<b>C</b>	Off	Off	Off	On	On	
<b>D</b>	On	Off	Off	On	On	
<b>E</b>	Off	On	Off	On	On	
<b>F</b>	On	On	Off	On	On	

## Troubleshooting

1. Check that the connection is correct and that all transmitters and receivers in the system are set to the same channel. If the flash alert is still not working, move on to the next steps.
2. *Set the flash alert to test mode:* Move the white lamp cover upwards in order to access the channel selector. Set microswitch 3 to ON. The yellow LED starts to flash.
3. *Check interference from other equipment:* The red LED now indicates whether there are other signals in the surrounding area which are transmitting on the same frequency as the flash alert. If the red LED does not light up or flashes 1-2 times per second it is OK. If, however, it stays on or flashes rapidly, this is a sign of interference.

*Action:* Move the flash alert to another location in the room.

4. *Check interference from another transmitter:* If the yellow LED is constantly on and the red and orange LEDs are flashing, this means that the flash alert is receiving interference from another transmitter in the vicinity which is transmitting on the same channel.

*Action:* Change the channel of the transmitters and receivers in the system (see “Changing channels” above).

5. *Quit test mode:* Set microswitch 3 to OFF. Then take the mains adapter out and put it in again to prepare the flash alert for normal use. Then test the system.

## Cleaning and maintenance

Vacuum outside of Lynx Flash with a soft brush from time to time. Use a lightly moistened cloth to dry off the exterior. Never use cleaning products or solvents.

## Repairs

If the product malfunctions it must be sent to an authorised technician for repair. Do not attempt to open the case of the device since this would invalidate the warranty. If your product requires service, please contact your hearing care professional for assistance.

## Waste disposal

This product contains electrical or electronic equipment and should be disposed of carefully in the interest of your safety and the environment. Please contact your local hearing care professional concerning disposal of the product.

## General Precautions

- If you cannot see the Lynx Flash, or if you are asleep without using an under-pillow vibrator, you will probably not notice that the product is signaling an alarm. This is particularly important if there is a fire.
- Low-energy radiation (eg from burglary alarms, room surveillance systems, radio equipment, mobile telephones) will not damage the device.
- Warning: External devices connected to the product must be safe according the requirements of EN 60601-1-1, EN 60335-1 or EN 60065, as appropriate.

## Part number

BL-95 (including table stand) 200 30 78

### *Accessories:*

Analogue alarm clock 200 30 73

Under-pillow vibrator 200 30 40

## Technical data

Modulation:	AM.
Frequency:	433,92 MHz.
Number of channels:	16.
External input:	1-12 V DC. For alarm clock or aerial.
External output:	12 V DC. For under-pillow vibrator.
Lamp:	10 W 12 V halogen.
Power supply:	12 V AC/DC 600 mA.
Range:	Approx. 15 m indoors.
Dimensions:	140 x 70 x 45 mm.

Any issues relating to the EU Medical Device Directive 93/42/EEC should be directed to GN ReSound A/S.

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**INTERNATIONAL INQUIRIES:****GN ReSound AB ALD Division**