 **Lynx™** Vibranett Alarm Clock

ENGLISH: Analogue alarm clock for under-pillow vibrator.....page 2
NEDERLANDS: Analoge wekker met kussenvibratorPag. 4
NORSK: Analog vekkerklokke for putevibrator side 6
SVENSKA: Analog väckarklocka för kuddvibrator sid 8

Ordering number: 200 30 73

Analogue alarm clock for under-pillow vibrator

Our analogue alarm clock alerts the user with both sound and an under-pillow vibrator. It can be used as a stand-alone wake-up system or together with our alerting system Lynx.

Batteries

The alarm clock is powered by one 1.5 volt battery, size C. The under-pillow vibrator is powered by mains current. In addition, there is an alkaline back-up battery (9 volt) for the under-pillow vibrator in the event of a power failure. Always use mains power for the under-pillow vibrator (a green LED should light up on the clock face) so as not to drain the back-up battery. NOTE: You should change the back-up battery once a year, see instructions on next page.

Connecting the under-pillow vibrator


To Lynx Tactum: Connect Tactum's under-pillow vibrator to the Tactum charger. Then, connect the alarm clock to the charger, using the enclosed connection cord.

As a stand-alone wake-up system: Connect the under-pillow vibrator into the socket on the backside of the alarm clock.

To Lynx Flash: Connect the under-pillow vibrator to Lynx Flash. Then, connect the alarm clock to Lynx Flash, using the enclosed connection cord.

Wake-up

At wake-up time, the alarm clock will ring and the under-pillow vibrator starts vibrating, until you turn it off.

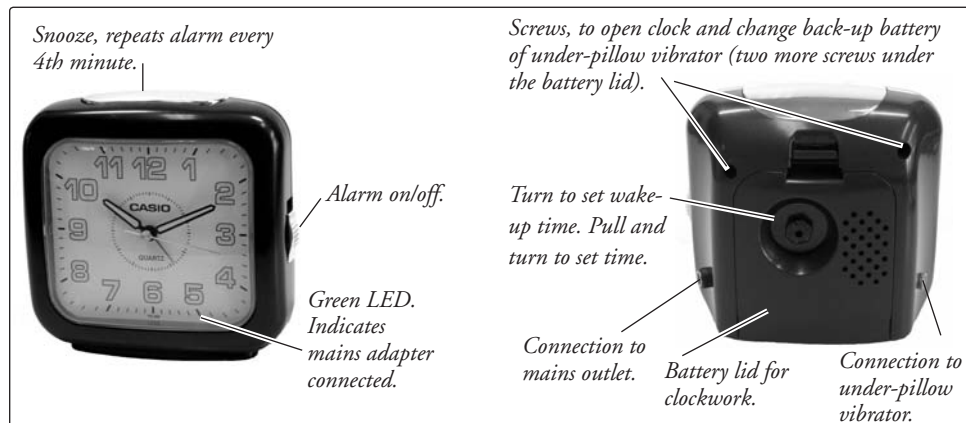Technical data

Power supply: 230 V AC 50-60 Hz.
Batteries: Alarm clock: 1 x 1,5 V (size C). Service life >3 years.
 Back-up battery for under-pillow vibrator: 1 x 9 V (6LR61). Service life 1 year.
Under-pillow vibrator: 9-24 V DC (80 mA).

Any issues relating to the EU Medical Device Directive 93/42/EEC should be directed to GN ReSound A/S.

Changing the back-up battery

1. Disconnect the mains cable from the clock.
2. Unscrew the four corner screws on the rear of the clock.
3. Pull out and remove the alarm set knob.
4. Carefully lift off the clock's rear cover so that you do not damage the cables.
5. Undo the 9 volt battery from the Velcro and disconnect the snap connector.
6. Connect the new battery in the same manner.

NOTE: Remember to reconnect the mains cable to the clock so that the green LED on the clock face lights up.

Troubleshooting

Type of fault	Cause	Action
The clock has stopped.	Flat battery.	Replace the old battery with a new 1.5 volt Size C battery.
The under-pillow vibrator is not working.	<ol style="list-style-type: none"> 1. The under-pillow vibrator is not connected to the clock. 2. The mains lead is not connected and the back-up battery is flat. 	<ol style="list-style-type: none"> 1. Connect the under-pillow vibrator to the clock. 2. Insert the mains cable and immediately change the back-up battery.
The clock does not ring.	The clock's alarm is set to OFF.	Set the clock's alarm to ON.

Cleaning and maintenance

Use a vacuum cleaner with a soft brush to clean the product on the outside. Use a damp cloth to wipe it on the outside. Never use cleaning products or solvents. From time to time check that the cables and connections are intact. The batteries last for at least 3 years. The back-up battery should always be changed once a year to ensure the clock will operate if there is a power failure.

Repairs

If your product malfunctions, it must be repaired by a qualified technician. If your product requires service, please contact your hearing care professional for assistance.

Waste disposal

This product contains electrical or electronic equipment and should be disposed of carefully in the interest of your safety and the environment. Please contact your local hearing care professional concerning disposal of the product.

General precautions

- The back-up battery should always be changed once a year to ensure the clock will operate if there is a power failure.
- Radiation from low-emitting equipment (burglar alarms, room surveillance systems, radio equipment, mobile telephones, etc) will not damage the device.
- Warning: External devices connected to the product must be safe according the requirements of EN 60601-1-1, EN 60335-1 or EN 60065, as appropriate.

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info@gnresound.nowww.gnresound.no**SVERIGE:***Postadress:***GN ReSound AB**
*Service:***GN ReSound AB ALD Division**
info@gnresound.sewww.gnresound-ald.se**INTERNATIONAL INQUIRIES:****GN ReSound AB ALD Division**
Device Directive 93/42/EEC:***GN ReSound A/S**