Transett 904

USER MANUAL FOR:

Transett 904 T Transett 904 R

Transett 904 TMS Transett 904 RM

Transett 904 TU Transett 904 C

ENGLISH page 2

SVENSKA.....sidan 13

NORSK.....side 24


A wireless communication system

Transett 904 is a wireless communication system, suitable in many situations, such as education with hearing-impaired students, conferences, guiding and interpretation. The system is very flexible and consists of different models of transmitters and receivers that can be combined in many different ways. In addition to this, many accessories are available, thus adding to the flexibility of the system.

Transmitters

There are three transmitters to choose from, all of them with a built-in microphone. Our transmitters have long operating time, making it possible to use the transmitter all day, without having to charge the battery.

Features Model	Non-locking switch	Built-in directional mic.	Miscellaneous
T			
TMS	X		Lightweight battery
TU		X	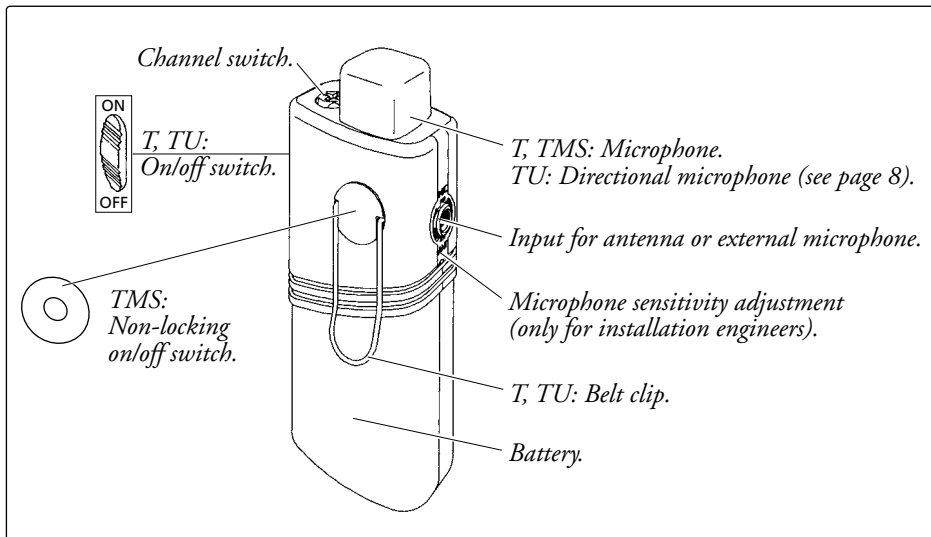


Receivers

There are three receivers to choose from, two portable and one wall-mounted. The equipment is designed to be used by hearing-impaired people and thus the sound quality is excellent in the whole frequency span.

Model \ Features	Built-in mic.	Miscellaneous
R		Portable
RM	X	Portable
Pedagog Phase Shift 90° *		Combined FM-receiver and loop amplifier.**

* You can also use our older model, Transett Pedagog III, in the system.

** For more information about Pedagog Phase Shift 90°, see its user manual.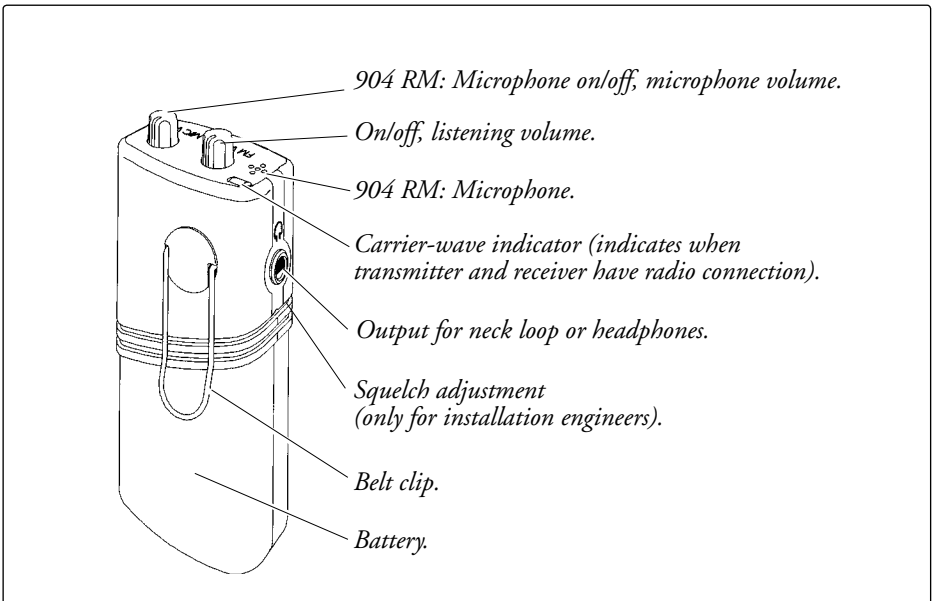

Examples of different user situations


The 904-system is very flexible and can be used in many situations. Here are a few examples.

Room with inductive loop

In a classroom, for example, with an installed inductive loop, the teacher uses the transmitter **904 T**. There is a wireless transmission of the speech to one of the antennas of **Pedagog Phase Shift 90°** *. The speech is amplified and transmitted by cable to the inductive loop. The students have their hearing instruments in the T-position, to be able to hear the sound from the loop.

When the students need to talk they use **904 TMS**. Their speech is transmitted to the other antenna of **Pedagog Phase Shift 90°** and then further to the inductive loop.


* The Phase Shift-functionality is not used in this example.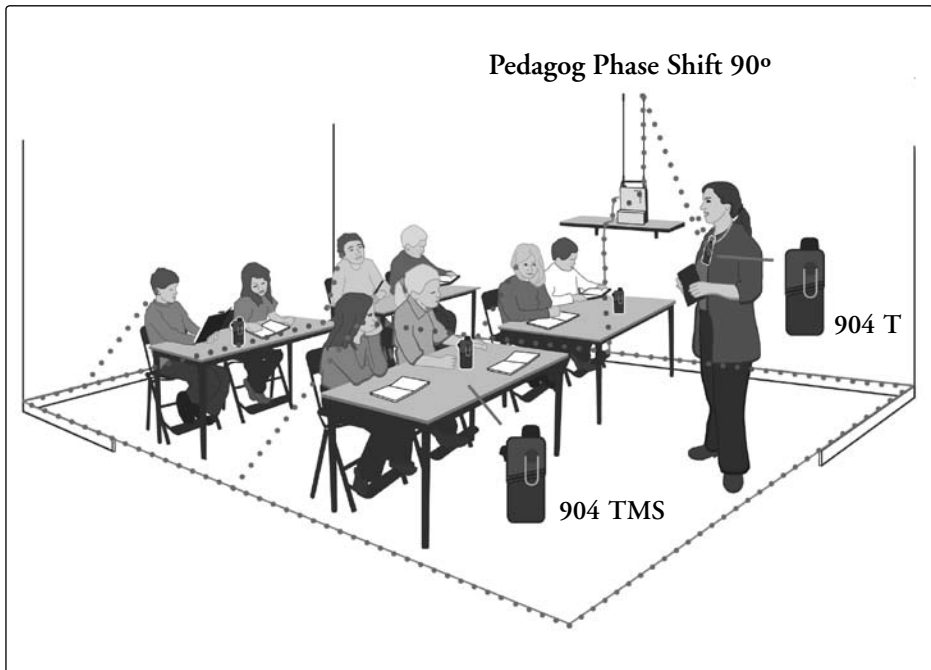

Room without inductive loop

In a normal classroom with an integrated hearing-impaired student, the teacher uses the transmitter **904 T**. The hearing-impaired student uses the receiver **904 RM** (with a built-in microphone). The student has a neck loop connected to 904 RM and his/her hearing instrument in T-position, to be able to hear the sound from the neck loop.

When one of the other students speaks, the hearing-impaired directs his/her 904 RM towards the speaking person. This solution is best for small classrooms.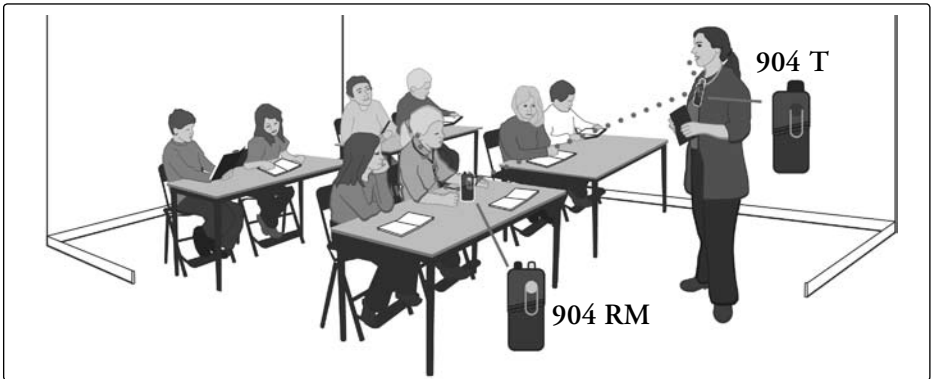

Guiding

The listeners can hear clearly, despite disturbing background noise and thus they can concentrate on what the guide says. Furthermore, it is possible to guide several groups at the same time, without them disturbing each other, simply by selecting different channels (frequencies). The guide uses the transmitter **904 TU** and the listeners each use a receiver **904 R** with headphones or neck loop.


Setting up the system

Setting channels


There is a wireless transmission of sound between transmitter and receiver in the 904-system. Transmitters and receivers must be set to the same channels in order for the transmission to work.

Setting channel on the transmitter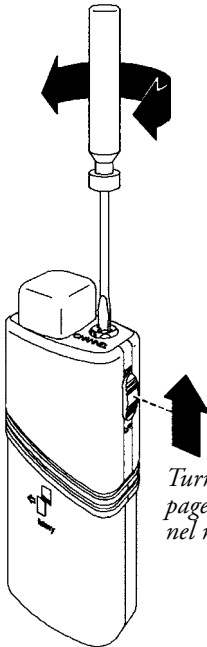

Turn on switch (see page 7) to see channel numbers.

Setting channel on the receiver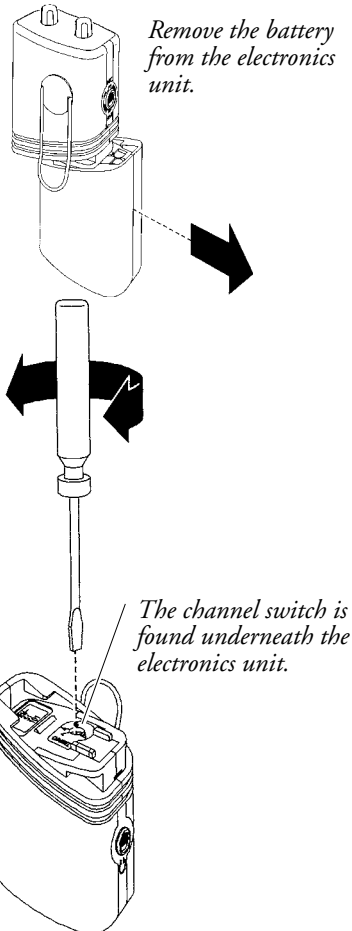

Remove the battery from the electronics unit.


The channel switch is found underneath the electronics unit.

How to wear the transmitter

You can wear the transmitter around your neck or in your pocket/belt. You can also place it on a table, if the person talking is close enough.

Around your neck


Use our neck loop, to carry the transmitter around your neck.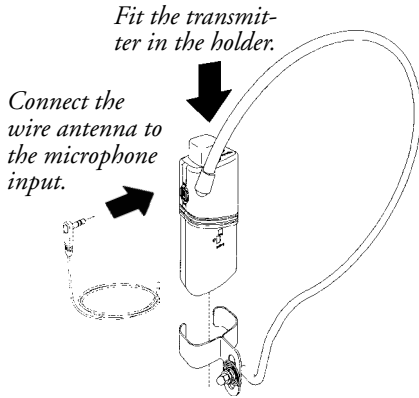

N.B: The wire antenna should always be connected when not using external microphone or recording cable.

In your pocket


When carrying the transmitter in your pocket, you need an external microphone (e.g. lapel microphone).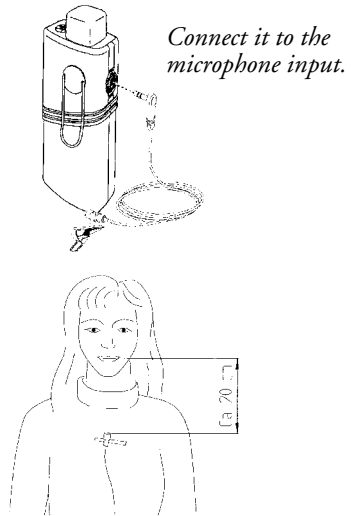

Operating the system


Turning on transmitter

T, TU: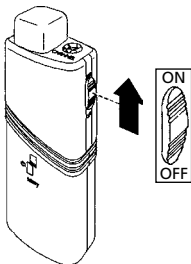


TMS:

Keep button pressed while talking.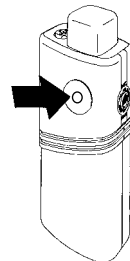



Directional microphone

904 TU has a built-in directional microphone suitable for noisy environments.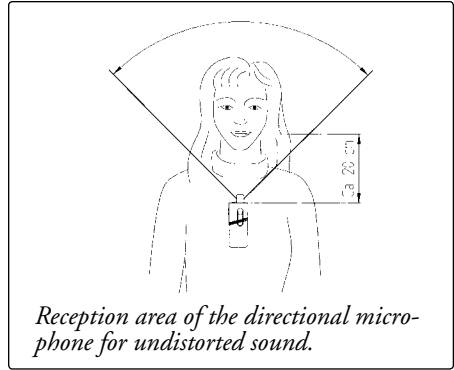

Listening with the receiver

To listen using the receiver, you connect a neck loop (hearing instrument users) or headphones to the output of the receiver. If the sound level is on high when turning on the receiver your hearing may be affected. Turn on the receiver and adjust the volume *before* putting on the neck loop/headphones.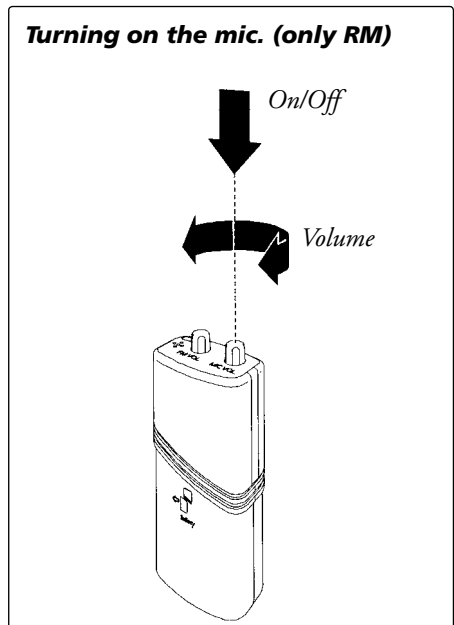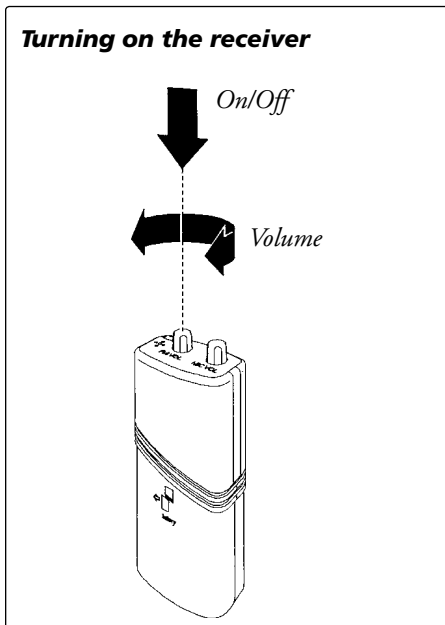

Charging transmitters and receivers


Transett 904 C is used for charging all transmitters and receivers in the 904-system. Transmitters and receivers are reversible in the charger (see drawing below).

Charging is time-controlled and it takes approx. 14 hours to charge a battery.

To reset the charger for a new charging period, the transmitter/receiver must be removed for at least 6 seconds. Even if the battery is fully charged, a new charging continues for 14 hours, before the charge indicator turns green.

Transett 904 C can be used as a single charger, but you can also connect up to 13 units. Transett 904 C can stand on the table or hang on the wall. Wall bracket, screws and plugs are enclosed.

You need a battery eliminator to Transett 904 C, and this is sold separately. For 1-2 units, you use Battery eliminator Mascot 8311 and for 3-13 connected units, you use Battery eliminator Transett 904 PS13.

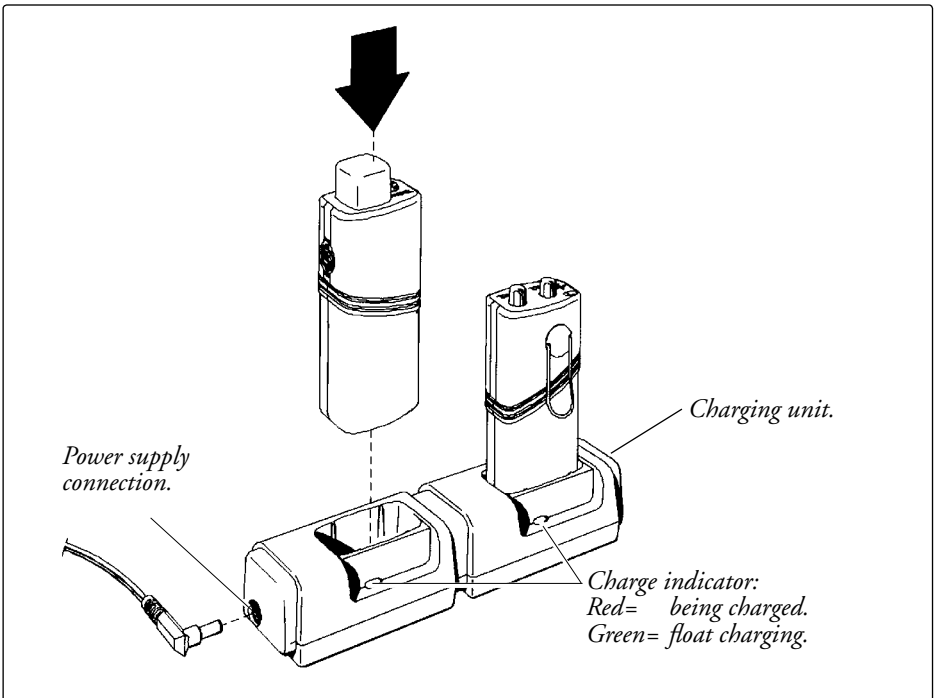Troubleshooting

Type of fault	Cause	Action
Receiver/transmitter not operational.	Unit is not switched on.	Switch unit on. The green lamp should light up (904T and 904R). Both lamps should light up (904 R/M).
No lamp goes on when the receiver/transmitter is switched on.	Battery is discharged.	Try to charge the unit for 15 minutes and repeat the test. If this works, the battery must then be charged up fully (takes about 14 hours).
No lamp goes on when 904 is charged.	Faulty contact.	Clean the battery contacts at the base of the battery and the contacts on the battery charger.
Battery discharges quickly.	Battery no longer operational.	Replace the battery.
No reception.	1. Wrong channel setting. 2. No antenna is connected.	1. Check that the transmitter and receiver are set to the same channel. 2. Check that the antenna (or microphone) is connected.
No sound (904 R/RM).	Volume control is set too low.	Turn volume up to a comfortable level.

Cleaning, safety and maintenance

Cleaning and maintenance

Use a vacuum cleaner with a soft brush to clean the product on the outside. Use a damp cloth to wipe it on the outside. Never use cleaning products or solvents. From time to time check that the cables and connections are intact. The battery has an operating life of about 500 charging cycles. Refer to page 9 for how to charge the battery.

Repairs

If your product malfunctions, it must be repaired by a qualified technician. Do not attempt to open the case of the device since this would invalidate the warranty. If your product requires service, please contact your hearing care professional for assistance.

Waste disposal

This product contains electrical or electronic equipment and should be disposed of carefully in the interest of your safety and the environment. Please contact your local hearing care professional concerning disposal of the product.

General precautions

- Low-energy radiation (eg from burglar alarms, room surveillance systems, radio equipment, mobile telephones) will not damage the device. They could, however, momentarily affect the sound quality or create noise in the device.
- Warning: External devices connected to the product must be safe according to the requirements of EN 60601-1-1, EN 60335-1 or EN 60065, as appropriate.

Ordering information & accessories

Frequencies and ordering numbers of transmitters and receivers vary according to country. For info, contact your local dealer.

Transmitters:*

Transett 904 T**
Transett 904 TMS
Transett 904 TU**

Receivers:

Transett 904 R
Transett 904 RM
Pedagog Phase Shift 90°

Complete 904-system: ***

Guide kits:

Case with 6 chargers and accessories
Case with 13 chargers and accessories

* Including two wire antennas.

** Including neck hook.

*** A complete 904-system delivered in a case containing the following parts: Transett 904 T, Transett 904 RM, neck hook, two wire antennas, neck loop with connection cord, lapel microphone, two chargers Transett 904 C and Battery eliminator Mascot 8311

Accessories:

T-models:

Lapel microphone	411 03 30
Headset microphone 3-in-1, 904	411 03 39
Headset microphone, noise cancelling, 904	411 03 43
Headset microphone behind ears, 904	411 03 44
Wire antenna	411 04 49
Neck hook	414 02 10
Stand	414 05 42

R-models:

Neck loop	414 04 60
connection cord	411 04 50
Headphones	200 60 20
ear pads	200 20 54
head strap pad	200 20 72
Hearing protector headphones	413 07 04

Charging:

Transett 904 C (charger)	414 05 25
Battery eliminator Mascot 8311	704 83 11
Battery eliminator Transett 904 PS13	411 04 73
Battery pack 904 B	411 04 48
Battery pack 904 BL (lightweight battery)	411 05 11

Miscellaneous:

Recording cable	411 04 53
Case (incl. compartments for all parts)	414 10 03

Technical data

Transett 904 T (all versions):

Radio frequency:	35-50 MHz.
Output:	10 mW.
Channels:	16, PLL controlled.
Channel spacing:	200 kHz.
Modulation:	FM.
Pre-emphasis:	50 μ s.
Frequency swing:	Max ± 35 kHz, ± 15 kHz at 90 dB SPL to the built-in microphone. Microphone sensitivity adjustable ± 10 dB.
Frequency deviation:	<100 ppm.
Limiting:	Knee point at 15 kHz swing. Increases the input signal by 20 dB, increases the swing by about 4 dB.
Distortion:	<0,5% at 90 dB SPL, <3% at 115 dB SPL.
Microphone input:	3,5 mm stereo jack, non-standard connection: tip-signal, ring-signal, earth-earth.
Power source:	Rechargeable NiMH-celles with nominal voltage 6 V.
Charging time:	Approx. 14 hours.
Operating time:	<i>T</i> , <i>TU</i> : >12 hours. <i>TMS</i> : >3 hours.
Switch (904 TMS):	Non-locking.
Microphone type (904 TU):	Directional microphone.
Measurements:	120 x 46 x 25 mm.
Weight (including battery):	<i>T</i> , <i>TU</i> : Approx. 145 g. <i>TMS</i> : Approx. 80 g.

Transett 904 R (all versions):

Radio frequency:	35-50 MHz.
Channels:	4, crystal controlled.
Channel spacing:	200 kHz.
Modulation:	FM.
De-emphasis:	50 μ s.
Squelch attenuation:	>66 dB.
Frequency range:	100 Hz – 10 kHz, -3 dB rel. 1 kHz.
LF output level:	4 V max at 32 Ohm.
Distorsion:	1% at ± 15 kHz swing.
Output:	3,5 mm bridged stereo jack: tip-out (plus), ring-out (minus), earth-not connected.
Power source:	Rechargeable NiMH-cells with nominal voltage 6 V.
Charging time:	Approx. 14 hours.
Operating time:	>10 hours.
Microphone (904 RM):	Built-in, with volume control. Microphone sensitivity 75 dB SPL for 4 V out (max vol.).
Measurements:	112 x 46 x 25 mm.
Weight (including battery):	Approx. 145 g.

Transett 904 C:

Charging current:	30-35 mA.
Float charging:	7-10 mA.
Charging time:	Approx. 14 hours.
Time before new charging:	5-7 s.
Measurements:	84 x 46 x 34,5 mm.
Weight:	120 g.

CE Any issues relating to the EU Medical Device Directive 93/42/EEC should be directed to GN ReSound A/S.

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SVERIGE:*Postadress:***GN ReSound AB**
*Service:***GN Resound AB ALD Division**
INTERNATIONAL INQUIRIES:**GN ReSound AB ALD Division**
MANUFACTURER*in the sense of EU Medical**Device Directive 93/42/EEC:***GN ReSound A/S**