ReSound **modeX** Transmitter

ENGLISH page 2
DANSK side 12
NEDERLANDS..... Pag. 22
NORSK..... side 32
SVENSKA sidan 42

Ordering number: 4120853

Contents

1. Product information	3
1.1 modeX system	3
1.2 modeX Transmitter	3
1.3 Overview of modeX Transmitter	4
1.4 Inputs and outputs	5
2. Installation.....	6
2.1 Location	6
2.2 Connection to modeX Receiver.....	7
3. Setting the transmitter	8
3.1 Programming.....	8
3.2 Programming modes	8
3.3 Programming menu	9
4. Troubleshooting	10
5. Care and safety	11
6. Ordering information and accessories.....	11
7. Technical data for modeX Transmitter	11

1. Product information


1.1 modeX system

modeX is a wireless communication system, which is suitable for many different situations such as when teaching hard-of-hearing students. The system comprises different types of radio transmitters and receivers, which can be combined in different ways. modeX is compatible with our earlier school system (the 904 system) and corresponding products from other manufacturers. It is therefore easy to add just what you need to an existing system.


The modeX system can be used both with a fixed inductive loop and together with hearing instruments with FM receivers.

1.2 modeX Transmitter


modeX Transmitter is a stationary transmitter which, when connected to other sound sources (for example other modeX products), relays this information wirelessly to, for example, a hearing instrument with FM receiver.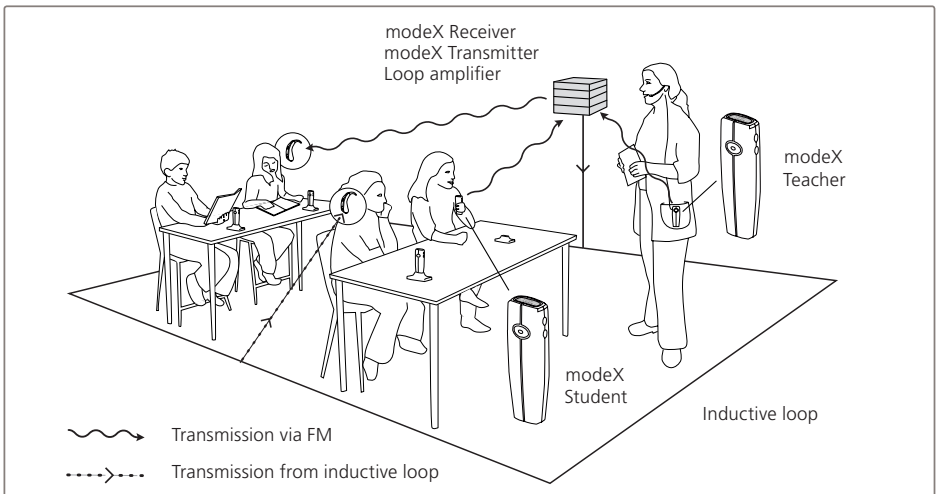

1.3 Overview of modeX Transmitter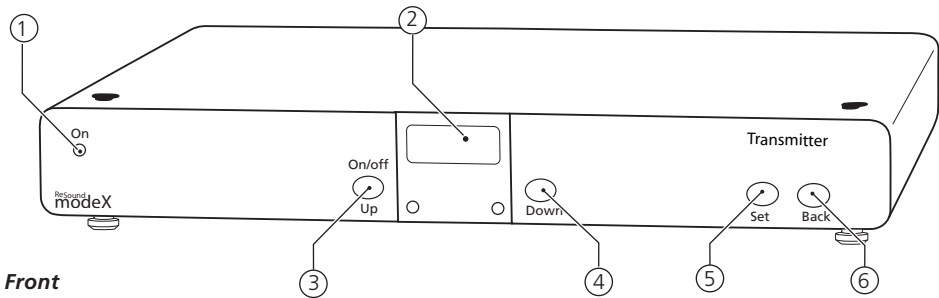

Front

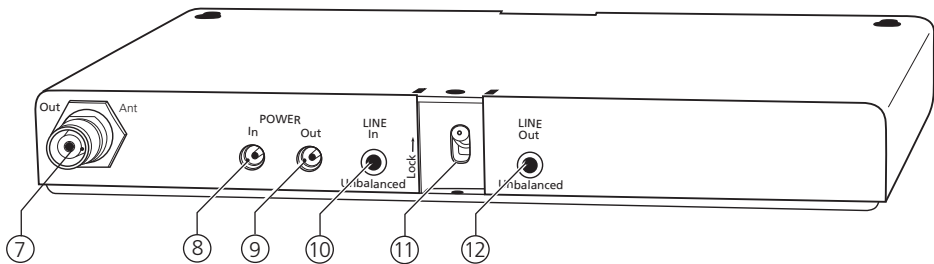Rear

Front

1. Green/red LED
2. Display

Programming buttons:

3. Up and On/Off button
4. Down button
5. Set button
6. Back button

Rear


7. Antenna signal, Out
8. DC in, 6 V
9. DC out, 6 V
10. Sound input (3.5 mm jack, unbalanced)
11. Locking button for display
12. Sound output (3.5 mm jack, unbalanced)

1.4 Inputs and outputs

Refer to the illustration on the previous page to see where the inputs and outputs are located.

Connector	Description
Ant Out	
TNC female	
POWER In/Out	
EIAJ class II	DC in: 6 Volt - — ⏏ + stabilised DC out: fused 3.15 AT
Line In/Out Unbalanced	
3.5 mm jack	Outer=earth, intermediate ring=signal, tip=signal. In: 1 stereo jack 3.5 mm. $2 \times 8.2 \text{ k}\Omega$, limiter-knee point approx. 700 mVac adjustable $\pm 15 \text{ dB}$. Out: 1 stereo jack 3.5 mm. Parallel with input. AC connected.

2.2 Connection to modeX Receiver

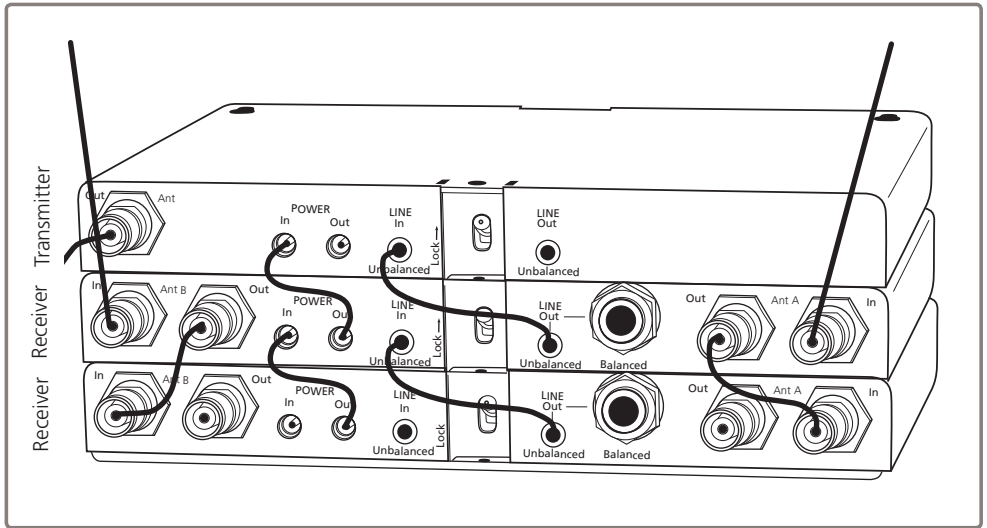3. Setting the transmitter

3.1 Programming

Set the locking button (11) to Off. Press the Set button (5) to access the programming menu.

Programming buttons

- | | |
|---------|--|
| Set | <ul style="list-style-type: none"> • Takes you to the next level. • Saves your settings. |
| Back | <ul style="list-style-type: none"> • Takes you back a level. • Quits programming mode. |
| Up/Down | <ul style="list-style-type: none"> • Increases/reduces a value. |

IMPORTANT When you have selected a setting, you must press Set to save it.

3.2 Programming modes

There are five different programming modes: Channel, Deviation, Line Level, On/Off key and Power.

Channel

Select channel: There are 106 pre-programmed channels to choose from. These are channels 20-99 which correspond to other channels in the modeX system. In addition, there are 19 H, J and N channels specifically for Phonak frequencies and 7 S channels specifically for Solaris frequencies. All channels can be received by modeX Receiver even if the frequency does not match precisely. Channels 00-19 do not have a pre-programmed frequency and can be adjusted as required, see below.

Change frequency: You can change channels 00-19 to any frequency. These channels do not work if no frequency value is selected. Hold in the button to step through more rapidly. The frequency steps through 25 kHz for each rapid press of the Up or Down button.

4. Troubleshooting

Type of fault	Problem	Action
No sound from the receiver.	<ol style="list-style-type: none"> 1. Is the correct channel set? 2. Is the modeX Transmitter diode off? 3. Is the Line In cable connected to the rear of the modeX Transmitter? 4. Is the deviation set correctly (often 5 kHz on hearing instrument receivers)? 	<ol style="list-style-type: none"> 1. Check the channel setting. See section 3.2, Channel. 2. Insert the power supply unit. Switch on the Transmitter (Up button). 3. Insert the Line In cable, in accordance with section 2.2 Connection. 4. See section 3.2, Deviation.
The sound quality is poor in the receiver.	<ol style="list-style-type: none"> 1. Is the modeX Transmitter deviation set to the same value as the receiver? 2. Are nearby channels causing interference? 3. Is the modeX Transmitter RF Output (transmitter power) set too low for the receiver? 4. Is modeX Transmitter too close to modeX Receiver? 	<ol style="list-style-type: none"> 1. See receiver user manual. 2. Check the channel setting. See section 3.2, Channel. 3. See receiver user manual. 4. Move the Transmitter further away or use a dipole antenna.
The sound is weak when I speak into the microphone.	<ol style="list-style-type: none"> 1. Is the volume on the receiver set correctly? 2. Is the Line level on the Transmitter set correctly? 	<ol style="list-style-type: none"> 1. See receiver user manual. 2. See section 3.2, Line level.

5. Care and safety

Cleaning and maintenance

Vacuum the outside of modeX Transmitter with a soft brush. Wipe the outside with a damp cloth. Never use cleaning products or solvents. From time to time check that the cords and connections are intact.

Repairs

If your product malfunctions, it must be repaired by a qualified technician. Do not attempt to open the case of the device since this would invalidate the warranty. If your product requires service, please contact your hearing care professional for assistance.

Waste disposal

This product contains electrical and/or electronic equipment and must be treated as electronic waste. The product must not be disposed of in household waste when it reaches the end of its life but taken to a recycling point or to the dealer. Please contact your local hearing care professional concerning disposal of the product.

General precautions

- Different types of radiation, e.g. from NMR or CT scanners, may damage the device. Therefore, do not wear the device in the vicinity of this type of equipment. Other types of radiation (burglar alarms, room surveillance systems, radio equipment, mobile telephones, etc) will not damage the device. These types of radiation could, however, momentarily affect the sound quality or create noise in the device.
- Mains adapter. Only approved mains adapters from GN ReSound may be used (see Accessories).

CE Any issues relating to the EU Medical Devices Directive 93/42/EEC should be directed to GN ReSound A/S.

6. Ordering information and accessories

modeX Transmitter	4120853
Cable, 3.5-3.5 mm plug, 0.10 m	4120859
Cable, DC-DC plug, 0.20 m	4120860
PS Universal, modeX (mains adapter, model: GT-41052-1506)	4120875
Antenna, dipole, 168-218 TNC	4120879

7. Technical data for modeX Transmitter

Radio frequency:	168-218 MHz.
Output power:	2-10 mW.
Deviation:	± 5, 15 kHz.
Type of modulation:	FM.
Frequency range:	250-20 000 Hz (-3 dB).
Distortion:	< 1,5% THD.
Signal/noise:	> 50 dBA at 5 kHz deviation.
Preset channels:	See www.gnresound-ald.com/modeX
Dimensions:	217 x 100 x 30 mm.
Weight:	1.2 kg.

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GN ReSound AB
GN ReSound AB ALD Division
INTERNATIONAL INQUIRIES:**GN ReSound AB ALD Division**